




4300 Dental Stool Series

Assembly Instructions




The Brewer Company, LLC
W165N5830 Ridgewood Dr., Suite 100
Menomonee Falls, WI 53051 USA

4335B & 4355B COMPONENTS

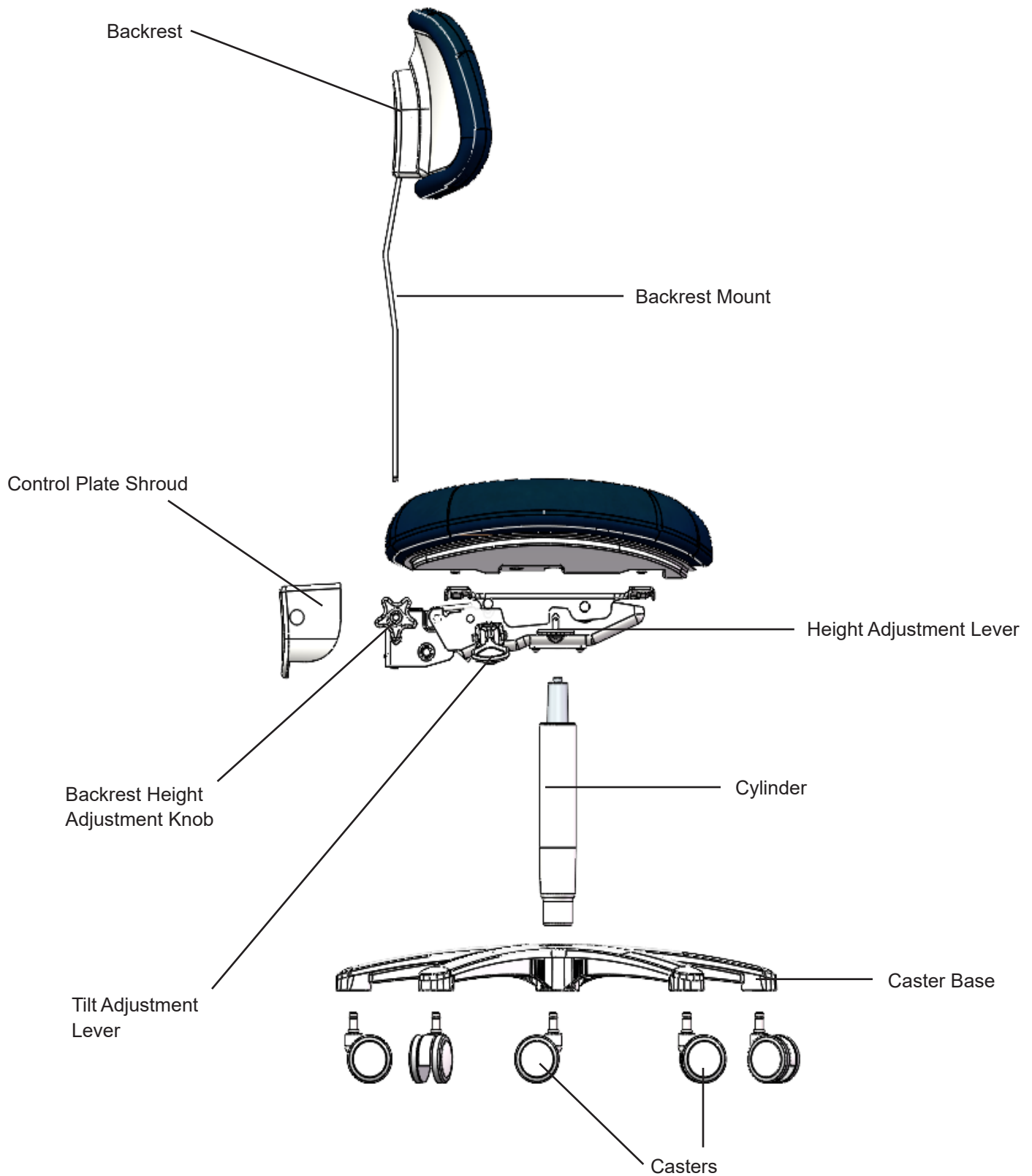

4345L/R, 4345BL/BR. & 4355BL/BR COMPONENTS

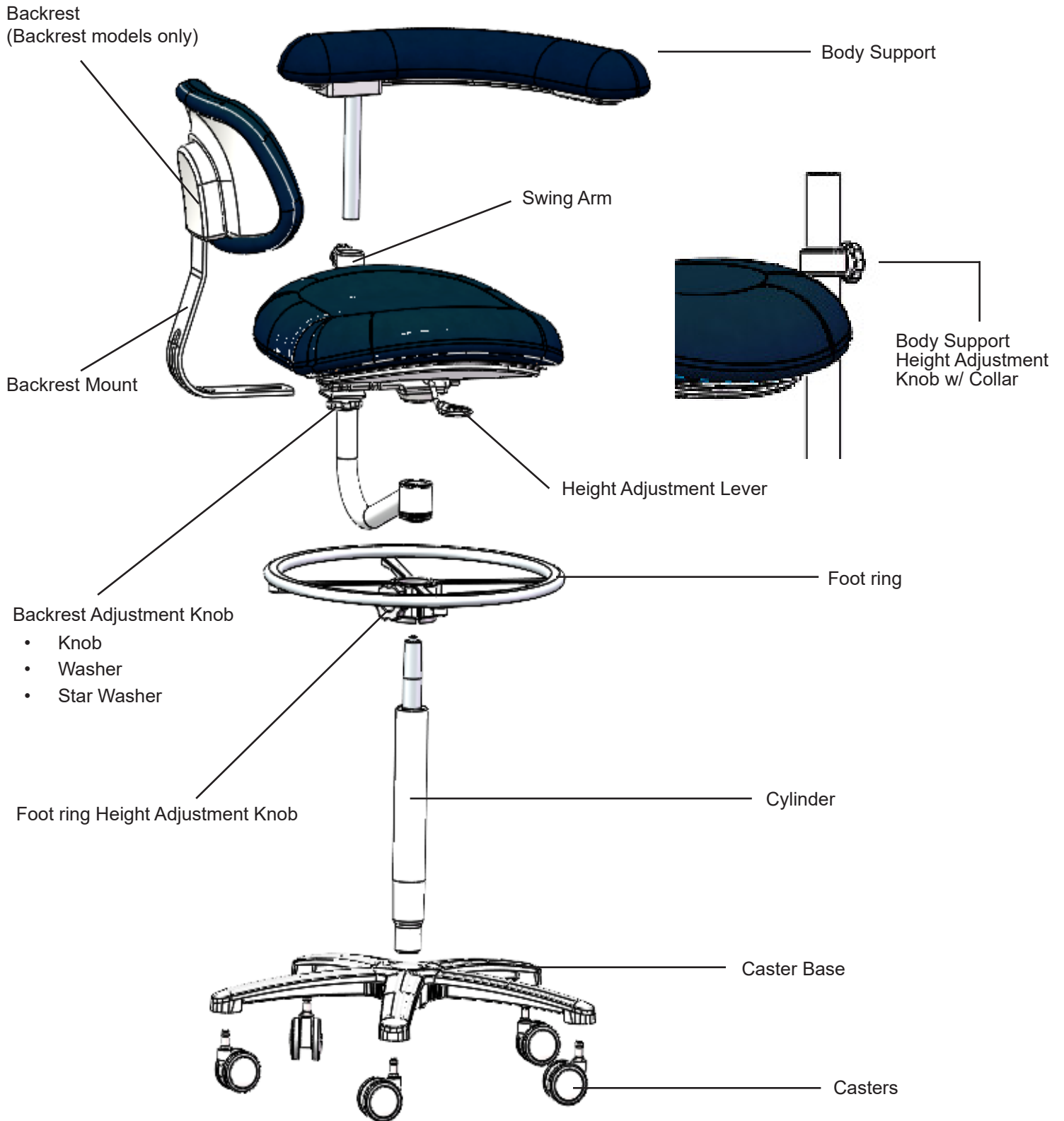

STOOL ASSEMBLY

1. Remove all parts from the carton and make sure all components are included for your model before assembling. Note: depending on your model, you may not have a backrest or foot ring etc.
2. Insert the (5) casters firmly into the opening on the stool base (Figure 1)
3. Set base on floor
4. Install the cylinder by placing the wider (lower) end into the center hold of the stool base (Figure 2)

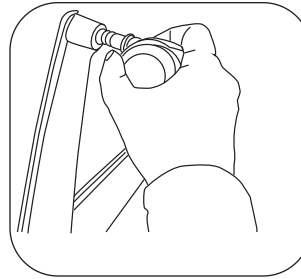


Figure 1

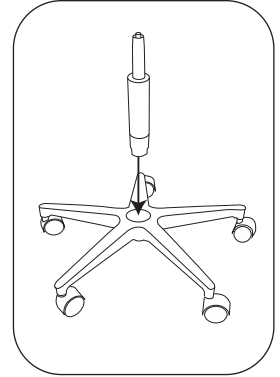


Figure 2

5. If a foot ring is included, loosen the hand knob, and slide the ring over the cylinder. Tighten knob at desired height (Figure 3)
6. If swing arm is included, lower it onto the spindle of the cylinder, and press down firmly until it contacts the locking ring on the spindle (Figure 4)
7. Place seat on cylinder, press down firmly (All models)

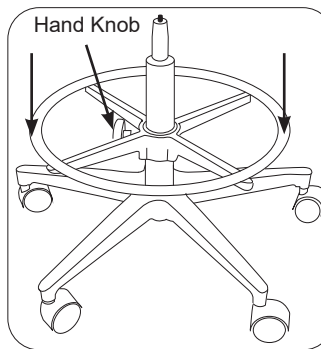


Figure 3

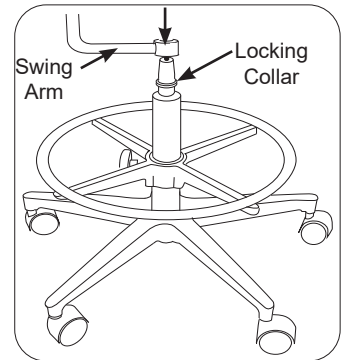


Figure 4

Installing Backrest

4335B & 4355B Operator Stool

1. Loosen knob and insert the backrest mount by sliding it through the slots in both the shroud and the control plate (Figure 5)
2. Secure the backrest: Tighten knob at the desired backrest height (Figure 5)

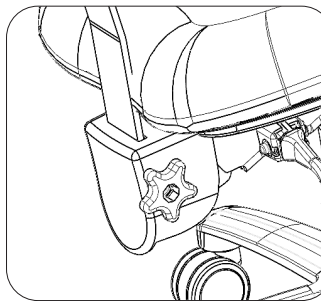


Figure 5

STOOL ASSEMBLY

Installing Backrest

4345BL/BR & 4355BL/BR Assistant Stool

1. Remove the hardware: Take off the knob, washer and star washer from the control plate (Figure 6)
2. Insert the backrest mount through the slot in the control plate (Figure 7)
3. Secure the backrest: Reinstall the washer, star washer and knob, then tighten to set the desired backrest depth (Figure 7)

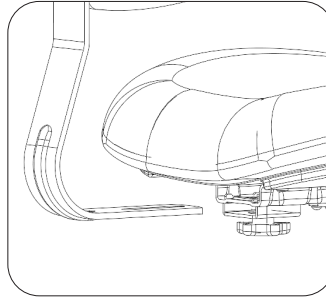


Figure 6

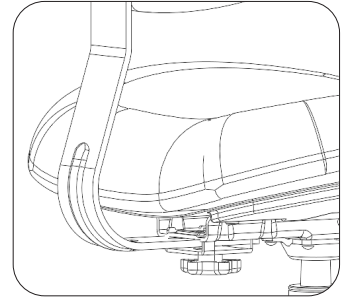


Figure 7

Installing Body Support (4345L/R, 4345BL/BR and 4355BL/BR)

1. Slide collar and insert the body support into the swing arm (Figure 8)
2. Ensure the collar and knob are lined up with the groove on the body support mount when inserting (Figure 8)
3. Tighten knob to desired height (Figure 8)

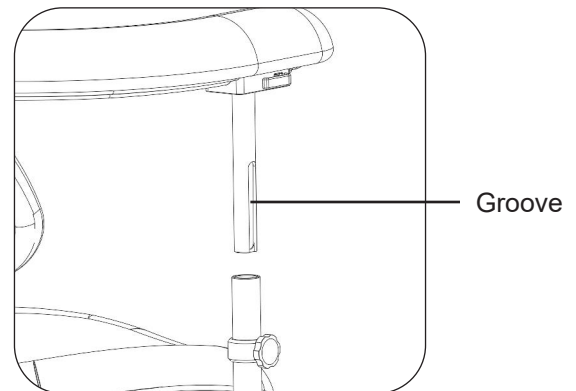


Figure 8

STOOL OPERATION

Height Adjustment (4345L/R, 4345BL/BR and 4355BL/BR)

1. To raise the seat: Slightly lift your weight off the seat while pulling up on height adjustment lever (Figure 9)
2. To lower the seat: Remain seated and pull up on the height adjustment lever (Figure 9)


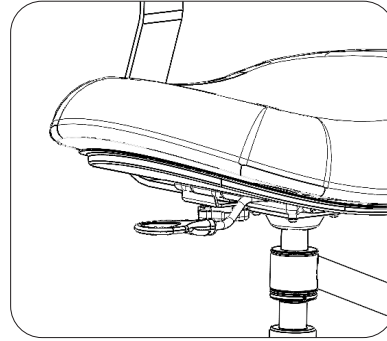


Figure 9

Height & Tilt Adjustment (4335B & 4355B)

1. To raise the seat: Slightly lift your weight off the seat while pulling up on height adjustment lever (Figure 10)
2. To lower the seat: Remain seated and pull up on the height adjustment lever (Figure 10)
3. To adjust the seat tilt, pull up on tilt adjustment lever and shift your body weight in the direction you want the seat to tilt (Figure 10)

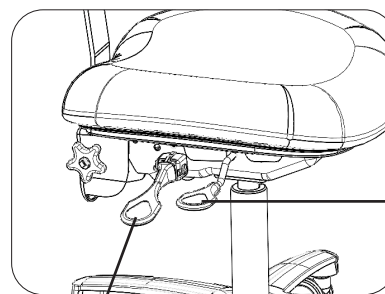


Height Adjustment

Figure 10

Seat Tilt

Body Support Adjustment (4345L/R, 4345BL/BR and 4355BL/BR)

1. To adjust the the body support, pull the lever to release the mechanism and move the support outward or inward as needed (Figure 11)
2. Releasing the lever engages the ratcheting mechanism, allowing the body support to lock into your preferred position (Figure 11)

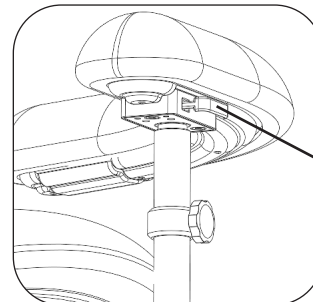

Lever

Figure 11